## ATTENTION

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## WARNING

### At reception:

- Charge the battery fully
- Sterilize the light guide
- Remove the protective caps sealing the hand piece and **store them** for use when cleaning the hand piece.


### Cleaning Instructions:

To clean the curing light or charging base, please carry out the following:

- Remove the light guide and sterilize it.
- Remove the eye shield.
- Unscrew the battery from the hand piece.
- Place the protective caps on both ends of the hand piece.
- Use a cloth or a wipe slightly damp with a non aggressive, pH-neutral disinfectant detergent and wipe the hand piece, battery, charging base, and eye shield.
- Never spray liquids directly onto the curing light or components. **Do not wipe the electrical contacts** or allow liquids to flow into the hand piece, battery, or charging base.
- Remove the protective caps from the hand piece and store them in a safe place.
- Screw the battery onto the hand piece.
- Attach the eye shield onto the hand piece. Apply a small amount of instrument lube on the seating area of the eye shield if you prefer a looser fit.
- Dry the light guide thoroughly and insert it into the hand piece.

Quaternary Ammonium based disinfectants are prohibited as they could damage your Mini L.E.D.

PREPARATION

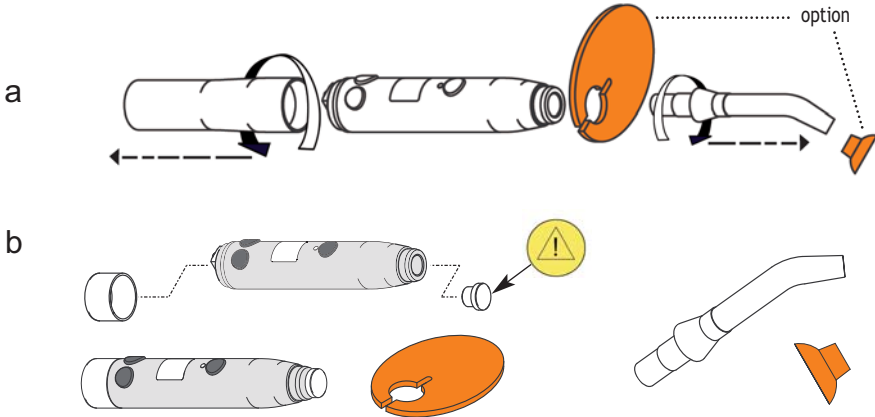CLEAN

CLEAN AND STERILIZE