


## BONDING TIPS

- Prepare teeth using standard office protocol.
- Firmly press adhesive into back of bracket.
- Seat on tooth with enough pressure to express perimeter adhesive “flash.”
- Remove adhesive flash and polymerize adhesive pursuant to your curing light instructions.
- Use a standard utility pliers to remove Visual Placement Aid (VPA) by its tail.



## DEBONDING TIPS

Debonding Radiance Plus is simple and predictable thanks to specially designed ceramic bracket debonding instruments.

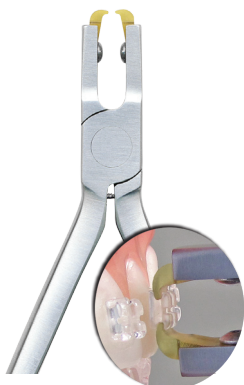

### Using Radiance Plus Debonding Pliers

REF 001-017405

This instrument provides a unique option for debonding Radiance Plus brackets. The tips are designed to grasp each bracket under the tie wing rather than making contact at the enamel/adhesive interface. These pliers are pre-adjusted with set screws to ensure that an optimal amount of pressure is applied to the bracket when the handles are squeezed together.

- Remove any remaining adhesive flash not removed at initial bonding with an debonding bur.
- Place the opposing tips of pliers occlusal/gingival under the tie wings.
- Apply constant pressure to the handles of the pliers.
- Gently move the angle of the pliers to the occlusal or gingival of the tooth with increasing force until the Radiance Plus bracket releases from the tooth.

REF 001-025385 Replacement tips (Qty 2 with wrench)  
Sterilize with autoclave



### Using Ceramic Bracket (Sushi) Debonding Pliers

REF 001-017403

This unique debonding instrument is specially designed for removing ceramic brackets. Only one edge of each blade is sharpened to ensure optimal fit along the enamel/adhesive interface. Opposing edges are parallel when the debonding instrument is open 3 mm, which is the average width of a ceramic bracket. Leave arch wires and ligatures in place during debonding.

- Remove any remaining adhesive flash not removed at initial bonding with a debonding bur.
- Place debonding blades along the enamel/adhesive interface, occlusal/gingival for upper centrals, mesial/distal for all other brackets.
- Squeeze handles slowly until bracket releases from the tooth.



SCAN FOR




DEBONDING  
DEMONSTRATION


# Bracket Color Identification

Visual Placement Aids (VPA's) are color coded to each specific tooth.


## A McLaughlin, Bennett, Trevisi\* System

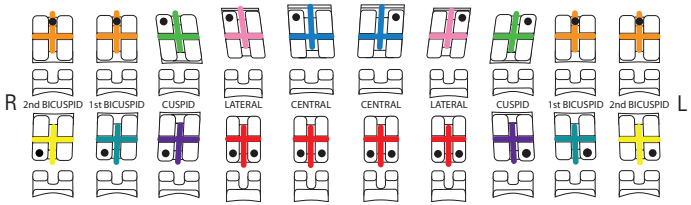

## A Roth\* System

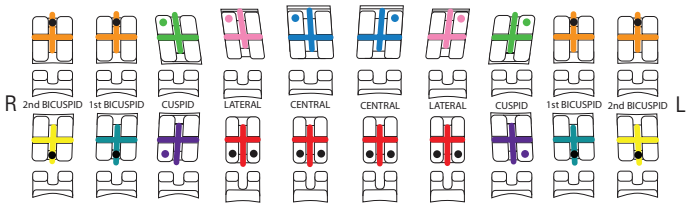

## The Alexander\* LTS System

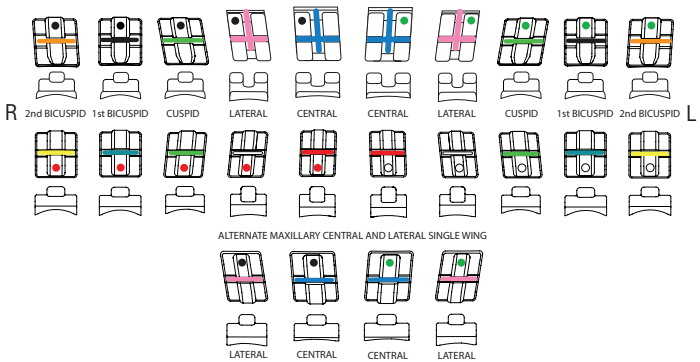

\* The American Orthodontics version of the Roth, McLaughlin, Bennett, Trevisi Systems do not claim to be a duplication of any other, nor does American Orthodontics imply that they are endorsed in any way by Drs. Roth, McLaughlin, Bennett, or Trevisi.

\*\* Dr. Wick Alexander is proud to endorse Alexander prescription brackets from American Orthodontics.



3524 Washington Avenue | Sheboygan, WI 53081 USA  
 +1 920 457 5051 | info@americanortho.com | www.americanortho.com

©2016 American Orthodontics Corporation All Rights Reserved

Made in USA | FOLLOW US ON    